

Administration in children (over 6 years old): the recommended dose is 1 to 4 ml of solution. The dose, duration and frequency of administration are prescribed by a doctor. Administration can be started with the minimal dose and if needed it may be increased.

Administration in adults (over 18 years old): use 1 container twice a day. If needed, the frequency of use can be increased up to 4 times a day.

Prepare the nebulizer for use.

1. Open the polymer bag and take the single-dose container out of it. Make sure the container is not damaged. Do not remove the container from the bag unless necessary.
2. Shake the removed container. Leave the other containers in the polymer bag and put them into the cardboard box.
3. Holding the container by its upper edge, rotate the other edge to open the container.
4. Insert the container into the nebulizer with the open edge down and give a slight press. Make sure that all the solution flew into the nebulizer.
5. Assemble the nebulizer and use it as intended.
6. After inhalation, it is required to rinse a mouth with water.
7. When using a face mask for inhalation, wipe the skin with a wet napkin to remove residual solution after the procedure.
8. Wash the nebulizer after usage, dispose the solution residues.

Adverse reactions

In persons with individual intolerance of the solution components, hypersensitivity reactions may occur. In individual cases, dizziness, hyperemia of mucous membrane, cough, bronchial spasm, throat irritation, nasal congestion, dry mouth, respiratory obstruction may appear.

In the event of any adverse reactions, the medical device should be discontinued immediately and the physician and manufacturer should be informed.

Limitations, precautions and warnings

- Solution is intended to be used only via inhalation.
- The first administration of a solution should be carried out under the care of a physician or qualified medical staff. Administration of solution by children should be performed under the supervision of adults.
- The safety of medical device application during pregnancy or lactation has not been studied, thus it is not recommended to use the device during pregnancy or lactation.
- Prior to application, verify the package integrity and check its shelf life. Do not use the device after the expiration date or if the package integrity is damaged.
- Do not mix with other solutions or medicinal products.
- For single use only. Repeated use can lead to infection. Do not re-use.
- Dispose according to the requirements of local disposal regulations.

Storage conditions

Store in a place protected from sunlight at a temperature of +5 °C to +30 °C. Keep out of the reach of children.

Shelf life

2 years. Shelf life is valid if the storage conditions are observed and the package is not damaged.

Authorized representative in the European Community

Diaco Biofarmaceutici S.R.L.
Via Flavia, 124, 34147, Trieste, Italy.
E-mail: info@diaco.it
www.diaco.it



Name and address of the manufacturer

Yuria-pharm LLC, 10, M. Amosova Str., Kyiv, Ukraine, 03038.

Tel.: +38 (044) 275-92-42, +38 (044) 275-01-08.

E-mail: uf@uf.ua

www.uf.ua

Manufacturing site address: 108, Kobzarska Str., Cherkasy, Ukraine, 18030.



If you have any comments on the medical device or would like to give us feedback, please use the following options to contact us:

- 1) email us at feedback@uf.ua;
- 2) send a text message via Viber, Telegram or WhatsApp to the number: +38 (095) 275-33-01;
- 3) call us at +38 (095) 275-33-01 or +38 (0800) 401-771 (charged in accordance with your operator's tariff plan).

Graphical symbols and their interpretation	
	Sterile medical device in primary packaging. Sterilized using steam or dry heat
	Do not re-use
	Consult instructions for use
	Manufacturer
	Temperature limit
	Date of manufacture
	Use-by date
	Batch code
	Mark of compliance with Directive 93/42/EEC on medical devices and the Notified Body number
	Authorized representative in the European Community
	Non-pyrogenic
	Do not use if package is damaged
	Do not re-sterilize

Date of last revision of instructions for use: 16.03.2023.

Version: 04.

